STEP-BY-STEP

ASSEMBLY GUIDE.

Medline Ultralight Transport Chairs

MDS808200F3R, MDS808200F3B, MDS808200F3BK,
MDS808200F3S, MDS808200F3P

What's Included:

A


B

C

D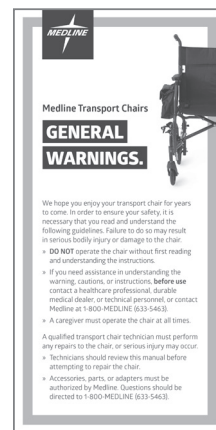

E

- A** Cup Holder
- B** Chair Frame with Wheels Attached
- C** 2 Footrests

- D** Step-By-Step Assembly Guide
- E** General Warnings

This guide applies to the following chairs:

Item Number	Description	Frame Color	Weight Capacity
MDS808200F3R	Ultralight Transport Chair, Red Aluminum Frame, Seat Width 19"	Red	300-lbs. (136 kg)
MDS808200F3B	Ultralight Transport Chair Blue Aluminum Frame, Seat Width 19"	Blue	300-lbs. (136 kg)
MDS808200F3BK	Ultralight Transport Chair Black Aluminum Frame, Seat Width 19"	Black	300-lbs. (136 kg)
MDS808200F3S	Ultralight Transport Chair Silver Aluminum Frame, Seat Width 19"	Silver	300-lbs. (136 kg)
MDS808200F3P	Ultralight Transport Chair Pink Aluminum Frame, Seat Width 19"	Pink	300-lbs. (136 kg)

First: Unfold the Seat Back

IMPORTANT NOTE:

Ensure the frame/handles and seat back are locked securely into place before using the chair.

Using the upper cane grip handles, push the seat back up until the release assembly spring buttons click fully into the folding mechanism on both sides. (You should be able to see the spring buttons fully engaged from the front of the seat back canes.)

Then: Open the Chair

With the seat in an upright position, place hands on either side of the arms of the chair and gently pull outward.

IMPORTANT NOTE:

Proceed with caution when opening the chair as there are several areas around the moving parts that could potentially pinch fingers.

As the chair begins to open, press down firmly on the outer edges of the seat with both hands, fingers flat.

Engaging the Wheel Locks

The wheel locks are pre-installed on the rear wheels.

Push the levers towards the front of the chair to lock the rear wheels.


Push the levers towards the back of the chair to unlock the rear wheels.


IMPORTANT NOTE:

Engage both wheel locks before entering or exiting the transport chair, when leaning forward, or while in an elevator or wheelchair lift.

Next: Install the Footrests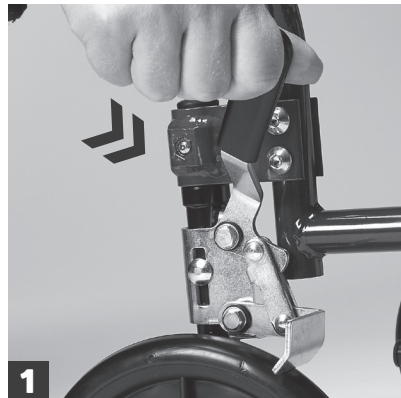


1 Engage the rear wheel locks by pushing the black toggle forward.

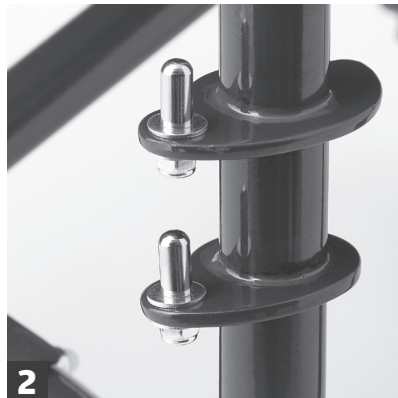2 Locate the two small silver pins on the chair above the front wheels.

3 Locate the two pre-drilled holes on the footrest.

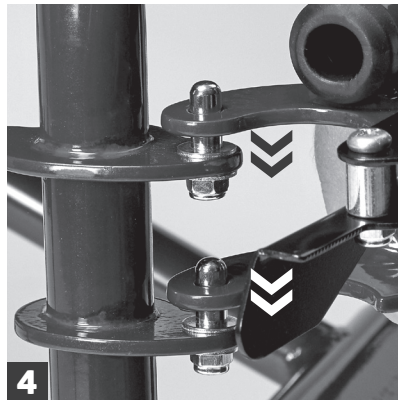4 Slide the footrest openings over the silver pins and place down.

5 Pull the footrest to the front of the chair until it locks into place.

6 Fold the foot plate down to use.

7 Repeat these steps on the other side.

To Swing Away the Footrest:

1 Fold the foot plate up to the vertical position.


2 Release the footrest lock by pushing the black lever forward.


3 Snap the footrest into the Stow-N-Go Clips located behind the front wheel to secure it to the chair.

To Adjust the Footrest Height: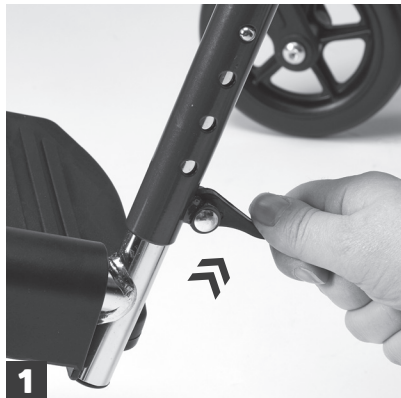

Flip up the black tab on the back of the leg rest.

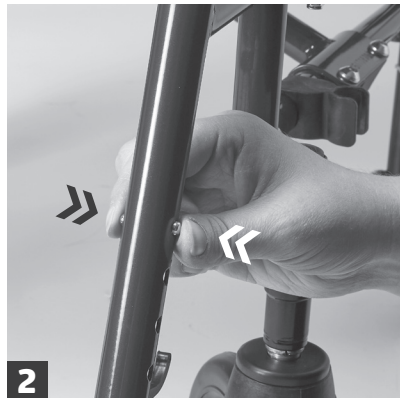Squeeze the push buttons on the stem of the leg rest to adjust the footrest height up or down.

When the desired height is achieved, release the pressure on the push buttons. The push buttons click securely into pre-drilled holes.

Flip down the black tab on the back of the footrest.

To Remove the Footrest From the Chair:

Disengage the tab lock.

Pull the footrest up and off of the silver pins.

Install Cup Holder

1 Unscrew the black plastic knob from the cup by turning it counterclockwise. Set aside.

2 Remove the clip.

3 Place the clip approximately 0.25" (0.6 cm) higher than the base of the chair arm. With the screw holes facing away from the chair, push the clip forward until it snaps into place.

4 Insert the cup screw through the pre-drilled holes.

5 Reattach the black plastic knob to the end of the screw and rotate clockwise to secure the holder to the chair.

Close and Fold

1 Fold the footrest plates up to a vertical position, pull them back, and snap them into the Stow-N-Go holders on the side of the chair.

2 Turn the front wheels back. Unlock the back wheels.

3 Find the black release tabs on each side of the chair back canes and pull down. The seat back will collapse.

4 Using both hands, grasp the two fabric handles on the seat upholstery and pull up.

5 Now grasp the arms of the chair and push inward until it collapses further.

6 Chair folds down for easy storage and transport.

GUIDE

D'ASSEMBLAGE

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WARNING:

This product contains a chemical known to the State of California to cause cancer.

Medline Industries, Inc.

One Medline Place, Mundelein, IL 60060

1-800-MEDLINE (633-5463)

medline.com | info@medline.com

FOLLOW US    

Some products may not be available for sale in Mexico or Canada. We reserve the right to correct any errors that may occur within this brochure.

© 2016 Medline Industries, Inc. All rights reserved. Medline is a registered trademark of Medline Industries, Inc.

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